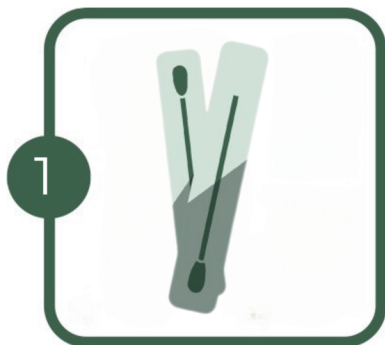
PAMĀCĪBA

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GenEra

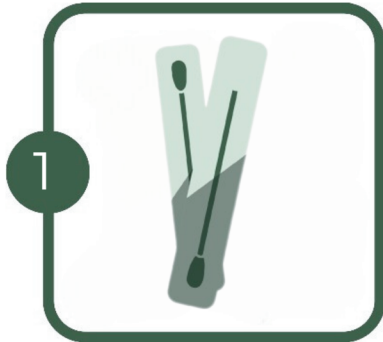
INSTRUCTION

FOR THE SAMPLING PRIMARY SAMPLES (ORAL MUCOSA BUCCAL SWABS)

ATTENTION!!!

- READ THESE INSTRUCTIONS TO THE END BEFORE TAKING SAMPLES.
- DO NOT COLLECT SAMPLES IMMEDIATELY AFTER BRUSHING TEETH, EATING (ESPECIALLY FOR INFANTS) OR SMOKING (WAIT AT LEAST ONE HOUR).


1
FOR EACH PERSON INVOLVED IN THE TEST YOU NEED TWO BRUSHES AND THEIR ORIGINAL INDIVIDUAL PACKAGING.


MANDATORY FILL IN THE TESTING REQUEST AND INDICATE:

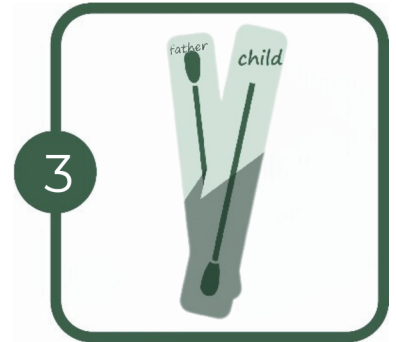
- NAME OR NICKNAME FOR ALL PERSONS INVOLVED IN TESTING;
- THE TYPE OF RELATEDNESS YOU WANT TO TEST (FOR EXAMPLE "PATERNITY" OR "HALF-SISTERS");
- CONTACT INFORMATION: YOUR PHONE NUMBER, E-MAIL.
THE FORM CAN BE FOUND ON THE WEBSITE WWW.GENERA.LV ON SECTION "USEFUL INFORMATION".

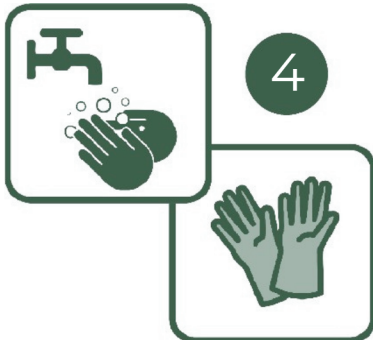
3
LABEL THE PACKAGE OF THE BRUSHES:
WRITE, FOR EXAMPLE, YOUR NAME OR NICKNAME, TERMS "FATHER" AND "CHILD" ALSO SUITS, SO THE LABORATORY CAN IDENTIFY SAMPLES OF DIFFERENT PERSONS.

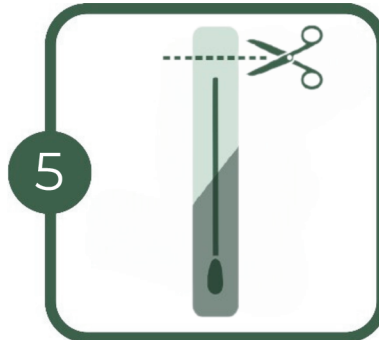
4
WASH AND DRY HANDS THOROUGHLY OR USE UNUSED DISPOSABLE GLOVES.

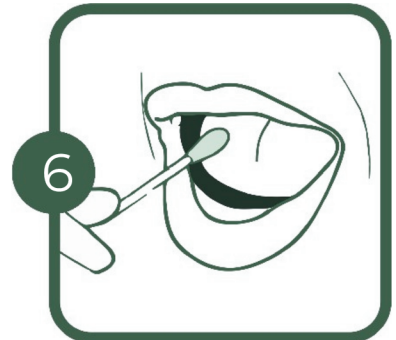
5
CAREFULLY OPEN AND TAKE OUT THE BRUSH FROM ITS PACKAGING.
PLEASE TOUCH ONLY THE STICK END AND DO NOT TOUCH THE BRUSH.

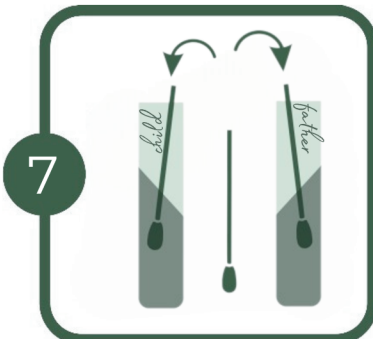
6
SCRAPE AND ROTATE THE BRUSH VIGOROUSLY ON THE INSIDE OF THE CHEEK FOR APPROX. **15 SECONDS.**

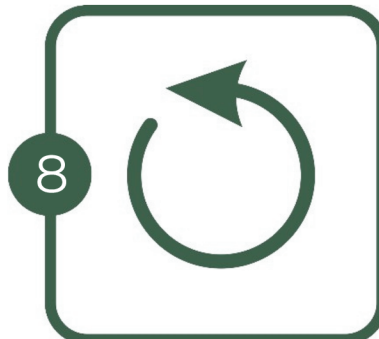7
IMMEDIATELY PLACE THE BRUSH INTO IN ITS ORIGINAL PACKAGING
WITH THE END CONTAINING THE SAMPLE (BRUSH) POSITIONED AT THE BOTTOM OF THE PACKAGE.

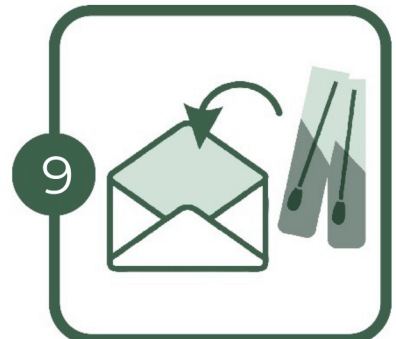8
REPEAT STEPS 5 TO 7 WITH THE SECOND BRUSH AND FOR OTHER PERSONS INVOLVED IN THE TEST.
NO SPECIAL CONDITIONS ARE REQUIRED FOR THE STORAGE OF THE SAMPLES WHILE THEY ARE PROTECTED FROM DIRECT SUN. FOR LONG-TERM STORAGE (LONGER THAN 2 DAYS), WE RECOMMEND PLACING THE SAMPLES IN THE FREEZER.

9
PLACE LABELED SAMPLES AND COMPLETED TEST REQUEST FORM IN THE CLEAN RETURN ENVELOPE. SEAL THE ENVELOPE.

NO SPECIFIC CIRCUMSTANCES ARE REQUIRED FOR SAMPLE MAILING IF THEY STAY PROTECTED FROM DIRECT SUN.

DELIVER SAMPLES TO THE LABORATORY / SEND BY MAIL. IF YOU USE THE ENVELOPE INCLUDED IN OUR EXTRAMURAL SAMPLE COLLECTION KIT, YOU DO NOT NEED POST STAMP FOR MAILING WITHIN LATVIA TERRITORY.

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