PAMĀCĪBA

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
INSTRUCTION

FOR THE SAMPLING PRIMARY SAMPLES (ORAL MUCOSA BUCCAL SWABS)

ATTENTION!!!

- READ THESE INSTRUCTIONS TO THE END BEFORE TAKING SAMPLES.
- DO NOT COLLECT SAMPLES IMMEDIATELY AFTER BRUSHING TEETH, EATING (ESPECIALLY FOR INFANTS) OR SMOKING (WAIT AT LEAST ONE HOUR).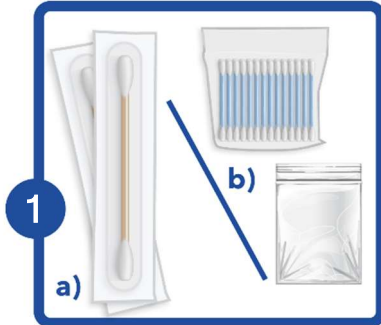

FOR EACH PERSON INVOLVED IN THE TEST YOU NEED:


- a) TWO COTTON SWABS (UNUSED HYGIENIC COTTON SWABS FROM NEW, UNUSED, UNOPENED PACKAGING);
 b) PLASTIC BAG (UNUSED, ORIGINAL PACKAGING OF INDIVIDUALLY PACKED COTTON SWABS CAN BE USED AS WELL).

OTHER ALTERNATIVE MATERIALS CAN BE USED AS SOON AS THEY ARE STERILE / UNUSED.

MANDATORY FILL IN THE TESTING REQUEST AND INDICATE:


-) YOUR NAME OR NICKNAME;
-) NAME OR NICKNAME OF OTHER PERSONS INVOLVED IN TESTING;
-) THE TYPE OF RELATEDNESS YOU WANT TO TEST (FOR EXAMPLE "PATERNITY" OR "HALF-SISTERS");
-) CONTACT INFORMATION: YOUR PHONE NUMBER, E-MAIL. THE FORM CAN BE FOUND ON THE WEBSITE WWW.GENERA.LV ON SECTION "USEFUL INFORMATION".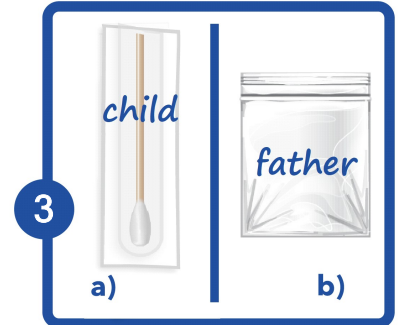


LABEL THE PACKAGE OF BUCCAL SWABS: WRITE, FOR EXAMPLE, YOUR NAME OR NICKNAME, TERMS "FATHER" AND "CHILD" ALSO SUITS, SO THE LABORATORY CAN IDENTIFY SAMPLES OF DIFFERENT PERSONS (IF YOU USE OUR EXTRAMURAL SAMPLE COLLECTION KIT, THE BUCCAL SWAB PACKAGING CAN BE ALREADY PRE-MARKED)


WASH AND DRY HANDS THOROUGHLY OR USE UNUSED DISPOSABLE GLOVES.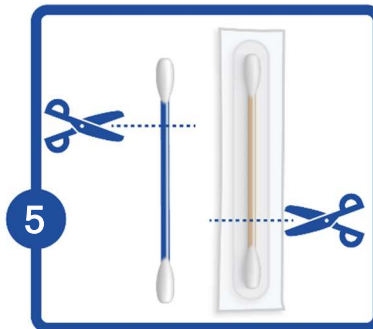


CAREFULLY CUT ONE END OF THE COTTON SWAB, **DO NOT TOUCH THE OTHER END** OR REMOVE THE SWAB CAREFULLY FROM ITS ORIGINAL PACKAGING (IF INDIVIDUALLY PACKED SWABS ARE USED). **YOU CAN TOUCH ONLY THE END WITHOUT COTTON WOOL.**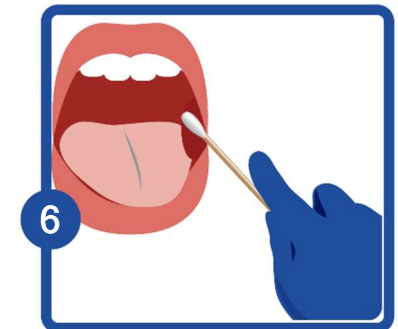

SCRAPE AND ROTATE THE SWAB VIGOROUSLY ON THE INSIDE OF THE CHEEK FOR APPROX. 15 SECONDS.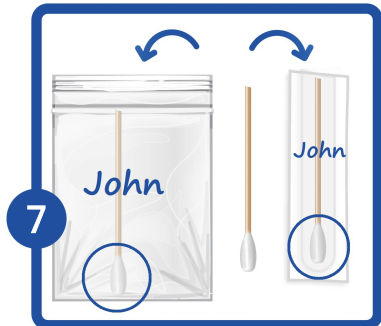

IMMEDIATELY PLACE THE SWAB INTO PREPARED BAG OR IN ITS ORIGINAL PACKAGE (IF INDIVIDUALLY PACKED SWABS ARE USED) **WITH THE END CONTAINING THE SAMPLE POSITIONED AT THE BOTTOM OF THE PACKAGE/BAG.**

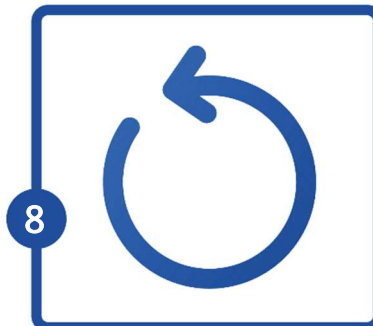REPEAT STEPS 5 TO 7 WITH THE SECOND COTTON SWAB AND FOR OTHER PERSONS INVOLVED IN THE TEST.

NO SPECIAL CONDITIONS ARE REQUIRED FOR THE STORAGE OF THE SAMPLES WHILE THEY ARE PROTECTED FROM DIRECT SUN. FOR LONG-TERM STORAGE (LONGER THAN 2 DAYS), WE RECOMMEND PLACING THE SAMPLES IN THE FREEZER.

PLACE LABELED SAMPLES AND COMPLETED TEST REQUEST FORM IN THE CLEAN RETURN ENVELOPE. SEAL THE ENVELOPE. NO SPECIFIC CIRCUMSTANCES ARE REQUIRED FOR SAMPLE MAILING IF THEY STAY PROTECTED FROM DIRECT SUN.

DELIVER SAMPLES TO THE LABORATORY / SEND BY MAIL TO: SIA GENERA, RĀTSUPĪTES STREET 1 K-1, RĪGA, LV-1067, LATVIJA. IF YOU USE THE ENVELOPE INCLUDED IN OUR EXTRAMURAL SAMPLE COLLECTION KIT, YOU DO NOT NEED POST STAMP FOR MAILING WITHIN LATVIA TERRITORY.

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