PAMĀCĪBA

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
INSTRUCTION

FOR THE SAMPLING PRIMARY SAMPLES (ORAL MUCOSA BUCCAL SWABS)

ATTENTION!!!

- READ THESE INSTRUCTIONS TO THE END BEFORE TAKING SAMPLES.
- DO NOT COLLECT SAMPLES IMMEDIATELY AFTER BRUSHING TEETH, EATING (ESPECIALLY FOR INFANTS) OR SMOKING (WAIT AT LEAST ONE HOUR).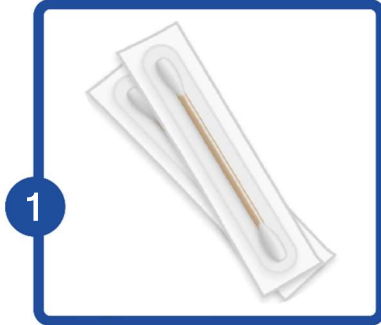


1

FOR SAMPLING YOU NEED:
a) TWO OR MORE COTTON SWABS;
b) PLASTIC BAG (UNUSED), OR ORIGINAL PACKAGING OF INDIVIDUALLY PACKED COTTON SWABS.


2

WASH AND DRY HANDS THOROUGHLY OR USE
 UNUSED DISPOSABLE GLOVES.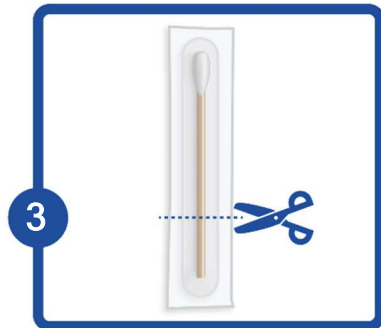


3

CAREFULLY REMOVE THE SWAB FROM ITS
 PACKAGING. **YOU CAN TOUCH ONLY THE END
 WITHOUT COTTON WOOL.**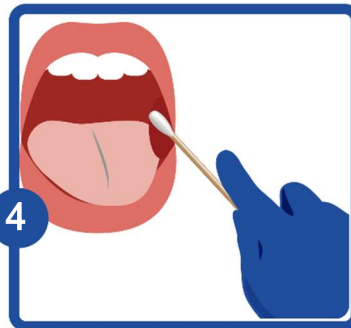


4

SCRAPE AND ROTATE THE SWAB VIGOROUSLY
 ON THE INSIDE OF THE CHEEK FOR APPROX. 15
 SECONDS.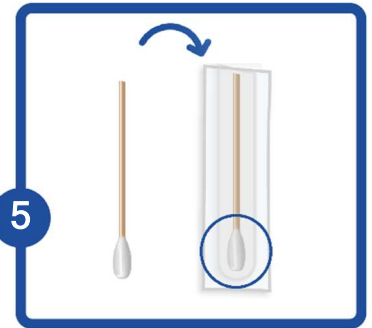

5

IMMEDIATELY PLACE THE SWAB INTO
 PLASTIC BAG OR IN ITS ORIGINAL PACKAGE
**WITH THE END CONTAINING THE SAMPLE
 (COTTON WOOL) POSITIONED AT THE
 BOTTOM OF THE PACKAGE/BAG.**

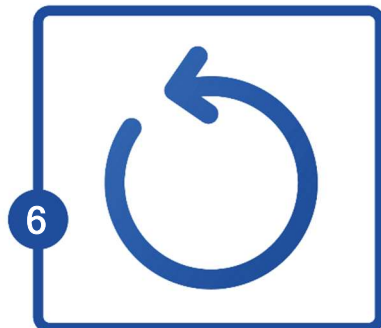
6

REPEAT STEPS 3 TO 5 WITH THE REST OF
 COTTON SWABS. NO SPECIAL CONDITIONS ARE
 REQUIRED FOR THE STORAGE OF THE SAMPLES
 WHILE THEY ARE PROTECTED FROM DIRECT SUN.
 FOR LONG-TERM STORAGE (LONGER THAN 2 DAYS),
 WE RECOMMEND PLACING THE SAMPLES IN THE
 FRIDGE OR FREEZER.

7

MANDATORY FILL IN THE TESTING REQUEST,
 INDICATE **YOUR PHONE NUMBER AND E-MAIL
 ADDRESS**. THE TESTING RESULTS WILL BE SENT TO
 PROVIDED E-MAIL ADDRESS, TO OPEN THE RESULTS
 YOU WILL NEED A PASSWORD, WHICH WILL BE SENT
 TO YOUR PHONE VIA SMS. THE RESULTS OF EACH
 INDIVIDUAL TEST MAY BE SENT AT DIFFERENT
 TIMES.

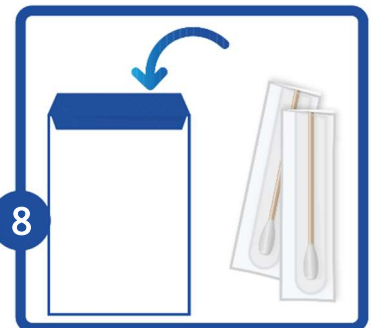8

**PLACE COLLECTED SAMPLES, COMPLETED
 TEST REQUEST FORM AND DOCTOR
 REFERRAL, IF ANY, IN THE RETURN
 ENVELOPE. SEAL THE ENVELOPE. NO SPECIFIC
 CIRCUMSTANCES ARE REQUIRED FOR SAMPLE
 MAILING IF THEY STAY PROTECTED FROM DIRECT
 SUN.**

**DELIVER SAMPLES TO THE LABORATORY / SEND BY MAIL TO: SIA GENERA, RĀTSUPĪTES STREET 1 K-1, RĪGA, LV-1067, LATVIJA.
 IF YOU USE THE ENVELOPE INCLUDED IN OUR EXTRAMURAL SAMPLE COLLECTION KIT, YOU DO NOT NEED POST STAMP FOR
 MAILING WITHIN LATVIA TERRITORY.**

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