PAMĀCĪBA

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
INSTRUCTION

FOR THE SAMPLING PRIMARY SAMPLES (ORAL MUCOSA BUCCAL SWABS)

ATTENTION!!!

- READ THESE INSTRUCTIONS TO THE END BEFORE TAKING SAMPLES.
- DO NOT COLLECT SAMPLES IMMEDIATELY AFTER BRUSHING TEETH, EATING (ESPECIALLY FOR INFANTS) OR SMOKING (WAIT AT LEAST ONE HOUR).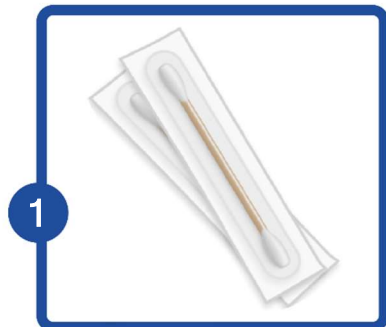

1


FOR SAMPLING YOU NEED:

- a) TWO OR MORE COTTON SWABS;
- b) PLASTIC BAG (UNUSED), OR ORIGINAL PACKAGING OF INDIVIDUALLY PACKED COTTON SWABS.


2

WASH AND DRY HANDS THOROUGHLY OR USE UNUSED DISPOSABLE GLOVES.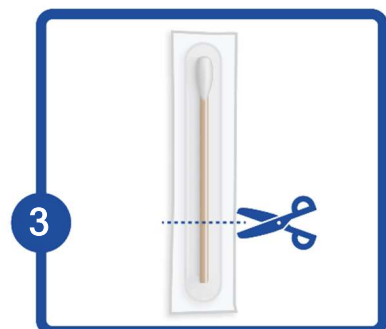


3

CAREFULLY REMOVE THE SWAB FROM ITS PACKAGING. **YOU CAN TOUCH ONLY THE END WITHOUT COTTON WOOL.**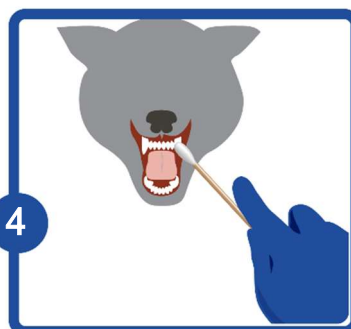


4

SCRAPE AND ROTATE THE SWAB VIGOROUSLY ON THE INSIDE OF THE ANIMALS CHEEK FOR APPROX. 15 SECONDS.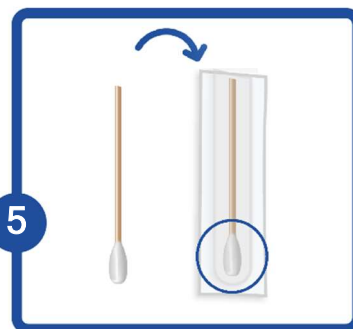

5

IMMEDIATELY PLACE THE SWAB INTO PLASTIC BAG OR IN ITS ORIGINAL PACKAGE **WITH THE END CONTAINING THE SAMPLE (COTTON WOOL) POSITIONED AT THE BOTTOM OF THE PACKAGE/BAG.**

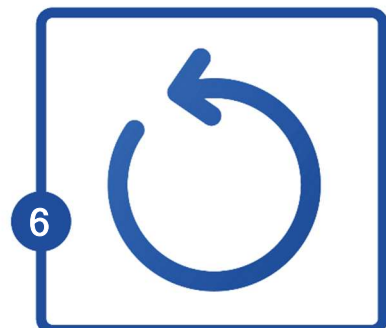
6

REPEAT STEPS 3 TO 5 WITH THE REST OF COTTON SWABS. NO SPECIAL CONDITIONS ARE REQUIRED FOR THE STORAGE OF THE SAMPLES WHILE THEY ARE PROTECTED FROM DIRECT SUN. FOR LONG-TERM STORAGE (LONGER THAN 2 DAYS), WE RECOMMEND PLACING THE SAMPLES IN THE FRIDGE OR FREEZER.

7

MANDATORY FILL IN THE TESTING REQUEST, INDICATE **YOUR PHONE NUMBER AND E-MAIL ADDRESS**. THE TESTING RESULTS WILL BE SENT TO PROVIDED E-MAIL ADDRESS, TO OPEN THE RESULTS YOU WILL NEED A PASSWORD, WHICH WILL BE SENT TO YOUR PHONE VIA SMS. THE RESULTS OF EACH INDIVIDUAL TEST MAY BE SENT AT DIFFERENT TIMES.

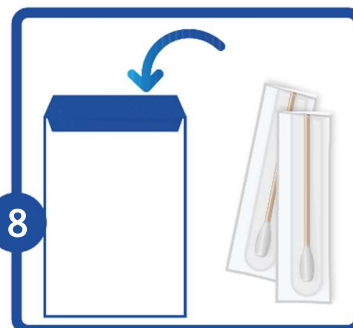8

PLACE COLLECTED SAMPLES, COMPLETED TEST REQUEST FORM AND DOCTOR REFERRAL, IF ANY, IN THE RETURN ENVELOPE. SEAL THE ENVELOPE. NO SPECIFIC CIRCUMSTANCES ARE REQUIRED FOR SAMPLE MAILING IF THEY STAY PROTECTED FROM DIRECT SUN.

DELIVER SAMPLES TO THE LABORATORY / SEND BY MAIL TO: SIA GENERA, RĀTSUPĪTES STREET 1 K-1, RĪGA, LV-1067, LATVIJA. IF YOU USE THE ENVELOPE INCLUDED IN OUR EXTRAMURAL SAMPLE COLLECTION KIT, YOU DO NOT NEED POST STAMP FOR MAILING WITHIN LATVIA TERRITORY.

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